

Medical Plaster - Quick pad wound dressing

- ▶ Latex-free
- ▶ Practical Smartbox

Features

- ▶ Does not stick to the wound
- ▶ Breathable material
- ▶ Water- and oil-resistant
- ▶ Permanently elastic
- ▶ Latex-free, therefore antiallergic
- ▶ Removable without leaving residue.

Benefits of use


- ▶ Joints remain freely movable
- ▶ Plaster, joint dressing, fingernail and fingertip dressing all-in-one
- ▶ Product adheres only to itself
- ▶ No cutting necessary because manually tear off

Areas of application

- ▶ Everyday injuries
- ▶ Smaller cuts
- ▶ Abrasions
- ▶ Protection against external abrasive influences to the skin



Packaging with Euro perforation

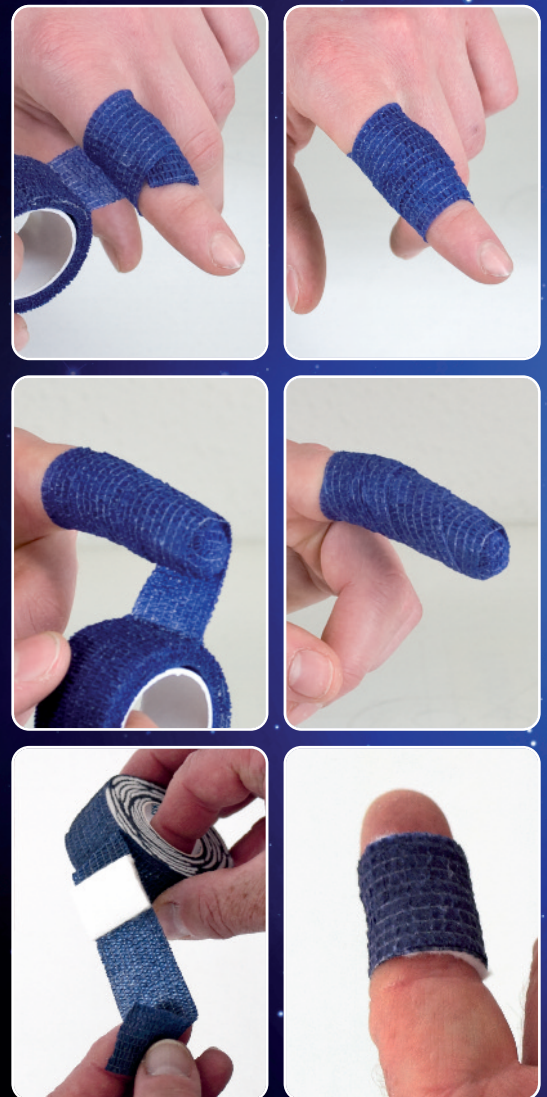

Medical Plaster - Quick pad wound dressing

Range of use

- ▶ Trade
- ▶ Household
- ▶ Kindergarten
- ▶ Sport
- ▶ Hobbies
- ▶ Pet Owner

Directions for use

Clean the wound from possible contamination.

Apply the **CareLine quick wound dressing** depending on the injury type (joint or tip injury) in accordance with the enclosed instructions. After wrapping the tape is easy to tear and to press firmly on the bandage.

Remove the **CareLine quick pad wound dressing** from the packaging and fold a wound dressing pad according to the size and bleeding of the wound. Place the wound dressing pad on the wound and wrap the dressing around it, applying moderate pressure on both sides. Lead the dressing back towards the wound dressing pad. Simply tear off and press firmly down on the end.

Caution

- ▶ In the case of incorrect or improper application, any product warranty becomes void.
- ▶ For professional and industrial use only.
- ▶ Please observe package insert!

Accessories

	Art.-No.
beko working gloves -cut-	905 020 2xx



Technical data

Basis	A composite nonwoven fabric		
Waterproof	Blood does not leak and the wound does not become wet. You can wash hands normally or work with wet hands.		
Stretchable	Stops excessive bleeding as a compress. It holds firmly when the joint is moved, does not interfere with work.		
Adhesive-free	Does not stick to the wound or to hair, but only to itself. No adhesive residue on your fingers. The dressing is very popular with children because it can be removed painlessly by rolling.		
Colour	beige / blue		
Storage	not above +25° C (+77° F), dry		
Contents/Packaging	Quick wound dressing 2 rolls 25 mm à 4,5 m (in a Smartbox)	12 pcs/box	Art.-No. 290 8 002
	Quick pad wound dressing 2 rolls 25 mm à 4,5 m (in a Smartbox)	12 pcs/box	Art.-No. 290 8 016